


GET AN EDGE ON NERVE PREPARATION

- Circumferential nerve constraint
- Independent, single-handed control
- Reliable and consistent tissue preparation

CHECKPOINT
EDGE™

NERVE
CUTTING KIT

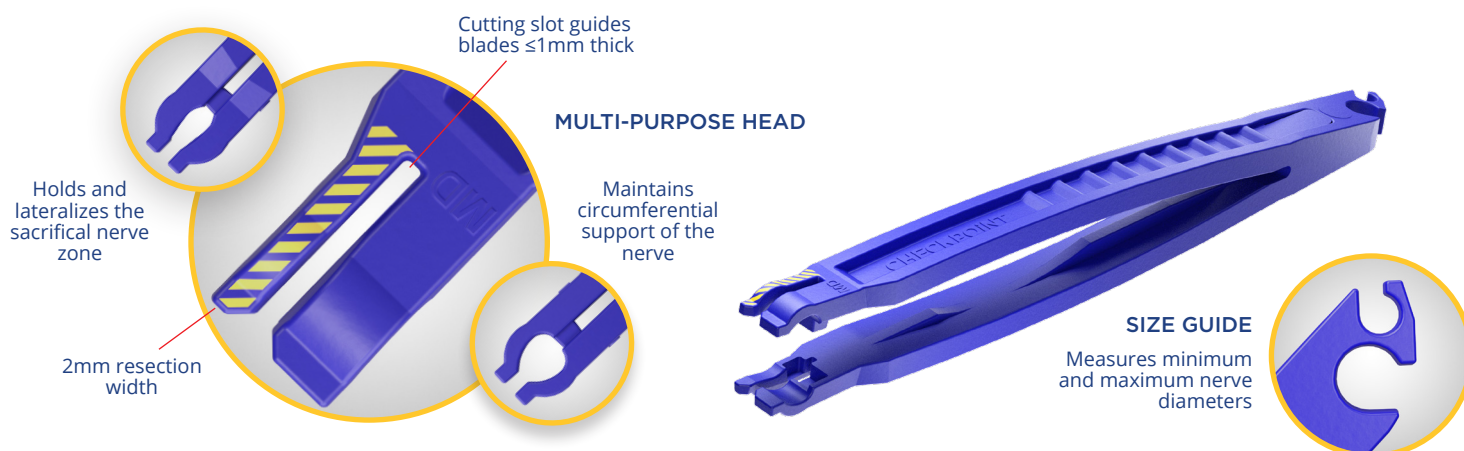

SUCCESSFUL NERVE REPAIR STARTS WITH PREPARATION

Traditional nerve cutting techniques may flatten and crush nerve tissue, leaving nerve ends misshapen and difficult to align. The CHECKPOINT EDGE™ Nerve Cutting Kit employs circumferential constraint to maintain the nerve's natural shape during transection. Constraining the nerve in this manner prevents nerve sliding, flattening and subsequent nerve edge malformation, preserving fascicular group alignment and tissue integrity.^{1,2}

EMPOWERING BETTER CONTROL

In addition to supportive constraint and nerve preservation, the Checkpoint Edge forceps are designed to empower surgeon independence. The head design permits a secure, single-handed hold that both grasps the sacrificial nerve zone to allow tensioning and gently constrains the nerve to be preserved. The cutting slot facilitates a right-angle cut with the other hand. Surgeon control of placement, tension, and transection reduces variability, resulting in reliable, consistent nerve tissue preparation for aligned coaptation.

REFERENCES

1. Smetana B, et al. Improving Nerve End Preparation for Neurorrhaphy through Use of a Circumferentially Constraining Surgical Device. Poster presented at 2018 ASPN Annual Meeting, January 12-14, 2018, Phoenix, AZ.
2. Jernigan EW, Patterson JMM, Draeger RW. How to cut a nerve: morphological implications of instruments used in preparation of severed nerves for neurorrhaphy. J Hand Surg Eur Vol. 2017 Nov;42(9):961-963.

ORDERING INFORMATION

Product name: CHECKPOINT EDGE™ Nerve Cutting Kit

Item number: 9250 | **Sold as a box of 4** | **Supplied sterile**

Kit includes (3) forceps: SM (1-3mm dia), MD (3-5mm dia), LG (5-7mm dia)

(1) microsurgical background | (1) SM69 scalpel blade | (1) 10R scalpel blade.

Handles not included.

Extra blades and handles available to order separately:

SM69 Scalpel blade, item no: V0001 (box of 25) | **Microsurgery handle**, item no: V0003 (1)

10R Scalpel blade, item no: V0002 (box of 100) | **No. 3 Scalpel handle**, item no: V0004 (1)

The Checkpoint Edge Nerve Cutting Kit is a single-use, sterile kit intended to provide surgeon aid in transecting nerve tissue for the purpose of nerve graft preparation, nerve repair, or removal of exposed nerve. Please see Instructions for Use for complete product specifications, indications, contraindications, precautions, and warnings.



NERVE CARE. **EMPOWERED.**

TF: 877.478.9106

O: 216.378.9107

F: 216.378.9116

info@checkpoint surgical.com

Checkpoint Surgical, Inc.
6050 Oak Tree Blvd., Suite 360
Independence, Ohio 44131

CS-9250-MKT-000-B